




EN ISO 11607

EN 868-8

**CASP**<sup>®</sup>

# Container for Plasma Sterilizer

Model: 603013

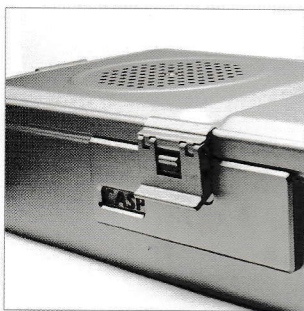




Manufacturer: MuDanJiang Plasma Physical Application Technology Co., Ltd.  
Address: JiangNan New Area, Dong'An District, MuDanJiang, HeiLongJiang Province, China  
Tel.: 82-2-830-4182  
Fax: 82-2-830-4183  
E-mail: [bjderun@naver.com](mailto:bjderun@naver.com)  
CASP is our company's registered trademark, without permission is strictly prohibited to use

[www.caspmedical.com](http://www.caspmedical.com)

Save running cost  
and resources

Observe indicator through  
visual window

No harm to  
instruments

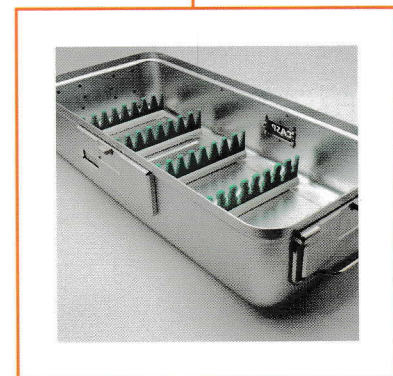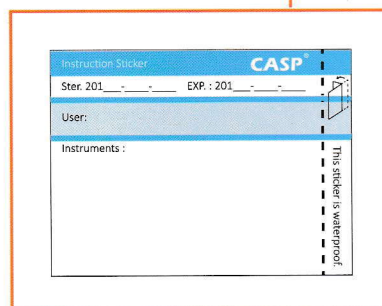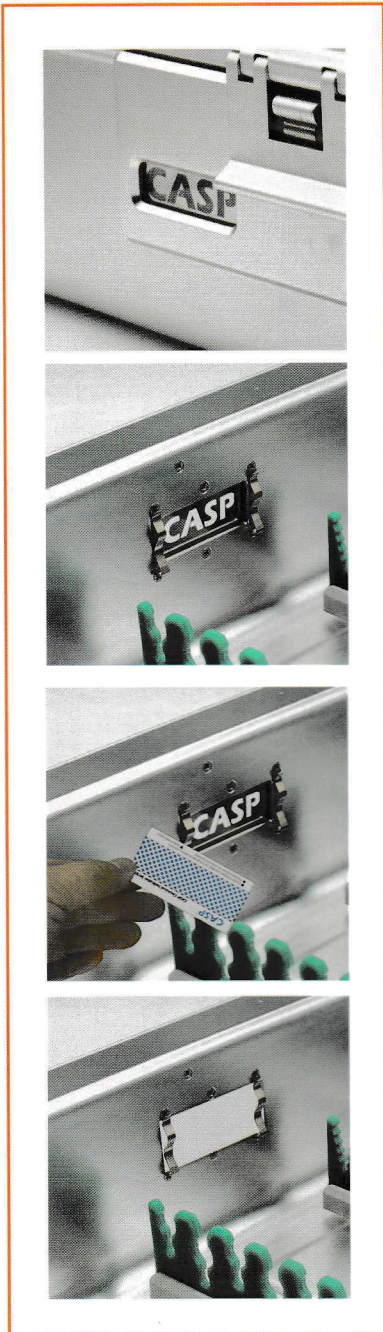

### Rapid Sterilization Check

No need to open the container for checking the result of sterilization. You can observe the indicator through the visual window.

### Instruction Sticker

Manual record of the sterilization date, expiry date, user and instruments in the container.


### Durable Construction

Fix the instruments with silicone bracket in the container.

Easy to pack

Guarantee the sterilization

Durable and lighter

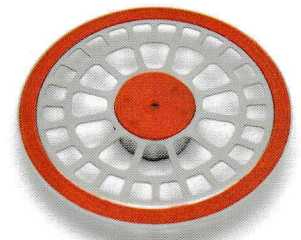

### Guarantee the Sterilization

We adopt American tyvek 1059B as Container Disk Membrane. Locking system prevents the container opening while transporting and ensures the sterilized condition.




### Locking System

Effective locking and nice feeling while using. Prevent the container opening while transporting and ensure sterilized condition.

Length(Outer/Inner)	Width(Outer/Inner)	Height(Outer/Inner)	Weight
643/586mm	313/256mm	134/110mm	4 kg


# Standard Accessories

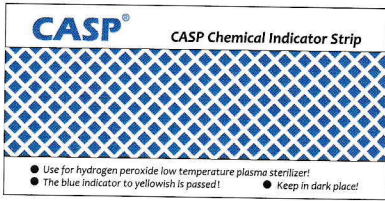

Chemical Indicator Strip



Disk Membrane

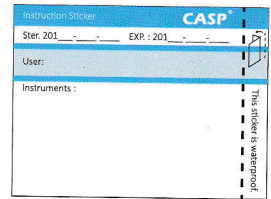


Instruction Sticker


# Accessories Information

## Chemical Indicator Strip

It is much more difficult to change the color of Chemical indicator strip(Class 5) than the Normal Biological Indicator. Under 55 C, the chamber volume is 96L/inject 1.5ml agent(H2O2)/after 180 sec., Normal Biological indicator was OK. But, Container Chemical indicator strip(Class 5) does not. After 10min., Container Chemical indicator strip(Class 5) was changed to yellowish. Container Chemical Indicator is only for single use.

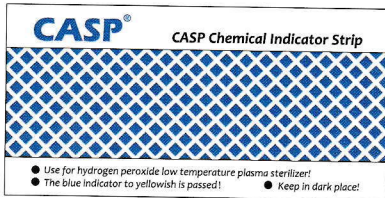


before use

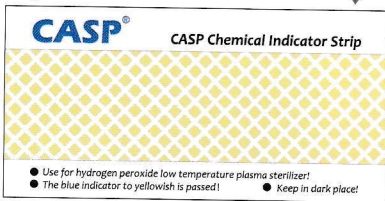

after use

## Disk Membrane

If the "not" on the Disk Membrane was disappeared, this Disk Membrane has been used.

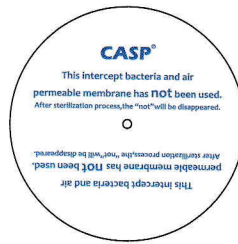


before use





after use

Test Result

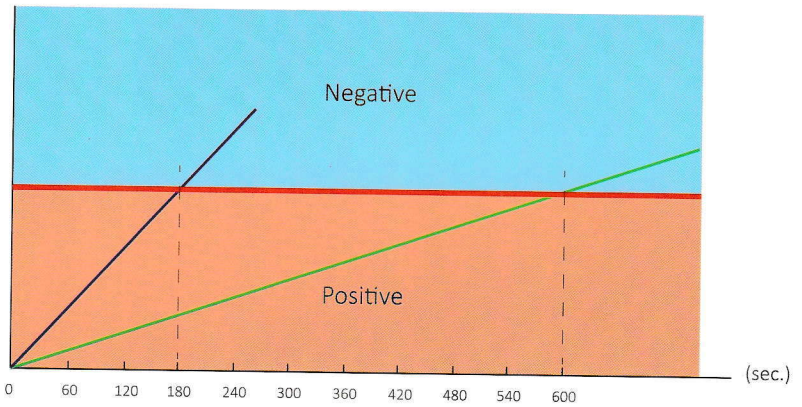

- Normal Biological indicator test result
- Container Chemical indicator test result
- Negative, qualified
- Positive, unqualified

# Test Report from CDC

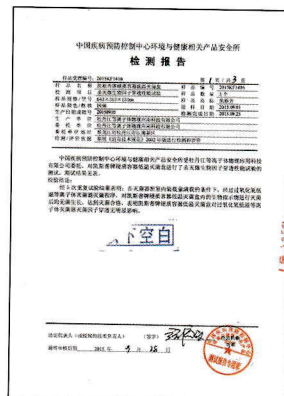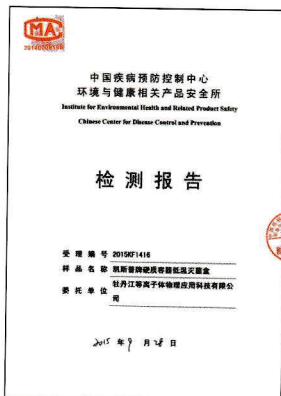

# CASP Trolley for Container

