amSafeX[™]

Urethra Safe Catheter

PREVENTS IATROGENIC URETHRAL INJURY



SIZES:

12 Fr | 14 Fr | 16 Fr | 18 Fr |

LENGTH:

16 INCH

BALLOON:

10-15 ML

MATERIAL:

SILICONE

COATED LATEX

SEE THE URINE
INFLATE THE BALLOON
PULL OUT THE OBTURATOR



amSafeX catheter provides a confirmed indication of balloon placement in the bladder, thus preventing urethral injury.



amSafex has two drainage eyes, distal eye 1 and proximal eye 2. In amSafex eye 1 temporarily remains blocked with an obturator connected to a thread which in turn is connected to pulling ring at the drainage port.

With eyel 1 blocked, urine will drain from eye 2 thus inidcating that balloon portion of catheterhas reached inside the bladder.

Post inflation of ballon, eye1 is unblocked by pulling out the obturator, unblocking urine drainage from eye 1.

Univlabs believes that cutting-edge, world-class medical devices should be available to all healthcare providers regardless of their clinic size or geographical disadvantage.

Sales Contact:

UnivLabs Technologies Pvt Ltd email: amsafex@univlabs.in

phone: +91-124-4116102|+91-9560925519

web: https://www.univlabs.in











UnivLabs Offices: Gurugram - INDIA | Lille - FRANCE

